

# CleanMount

## Quick Start Installation Guide

**STRONGARM**healthcare

Visit [strongarmhealthcare.com](http://strongarmhealthcare.com) to view our installation video.

### What's Included

- A) 1/4" Hex Key
- B) (4) Display Thumb Screws (M4 x 8)
- C) (4) Display Thumb Screws (M4 x 12)
- D) Rail cover


### Tools You'll Need

- #1 Phillips screwdriver
- Wire Cutter & Tie Wraps (for securing keyboard and mouse cabling: see Step 5 for instructions on accessing prewired USB hub)




### 1. Install Mounting Track\*

\* Refer to track installation instructions

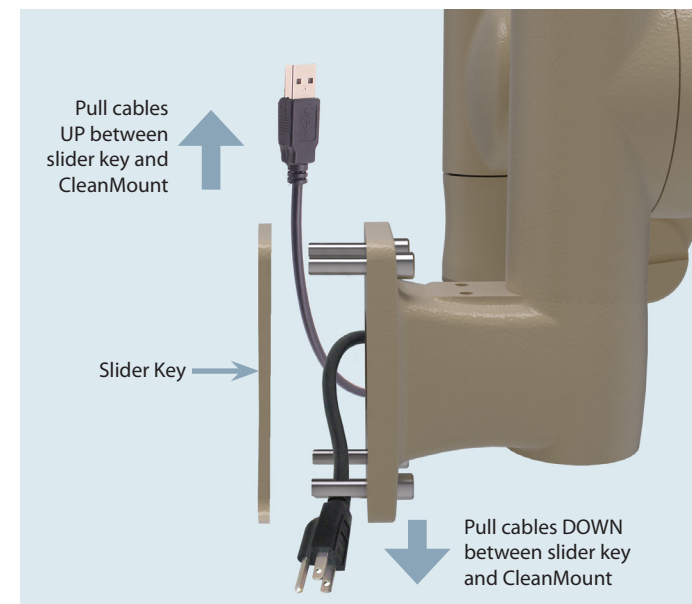


### 2. Orient Cables

1. Remove track slider key
2. Orient cables in the proper direction (up or down) and reattach slider key to CleanMount base

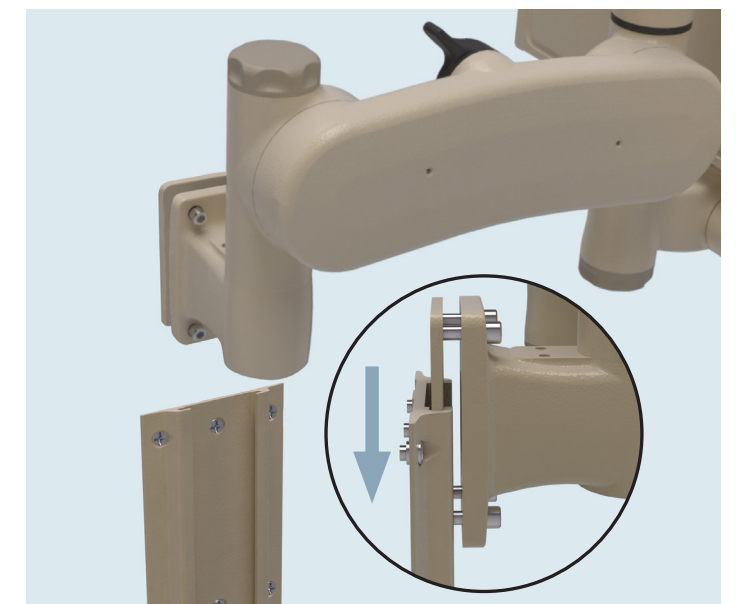


### 3. Install Onto Track

(Shown here without cabling)

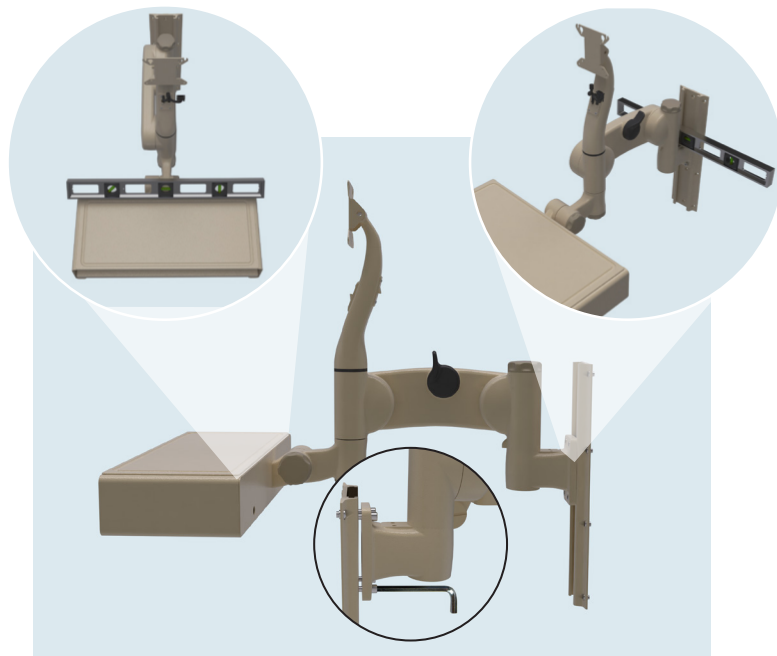


### 4. Level and Tighten

Use 1/4" Hex Key to Tighten Four Slider Key Bolts

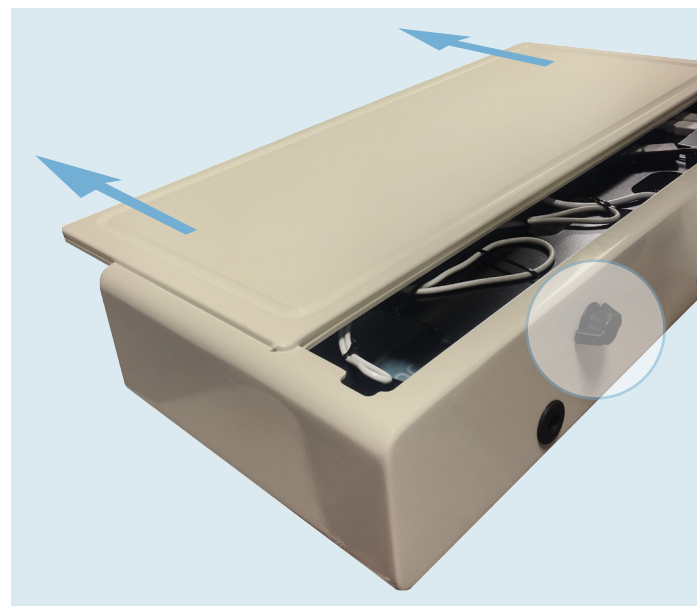


### 5. USB Hub Access

- 1) Turn latch on rear of keyboard enclosure to release cover. Pull cover forward to access the USB hub.

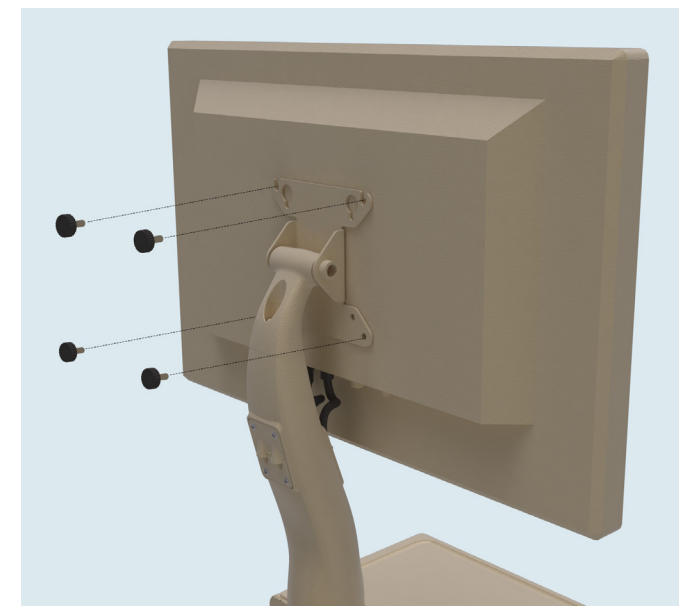

### 6. Install Display

Using (4) M4 x 8 Display Thumb Screws, attach display to CleanMount. (**NOTE:** If the M4 x 8 Display Thumb Screws can't be threaded sufficiently to provide a secure attachment, use the M4 x 12 Display Thumb Screws instead.)

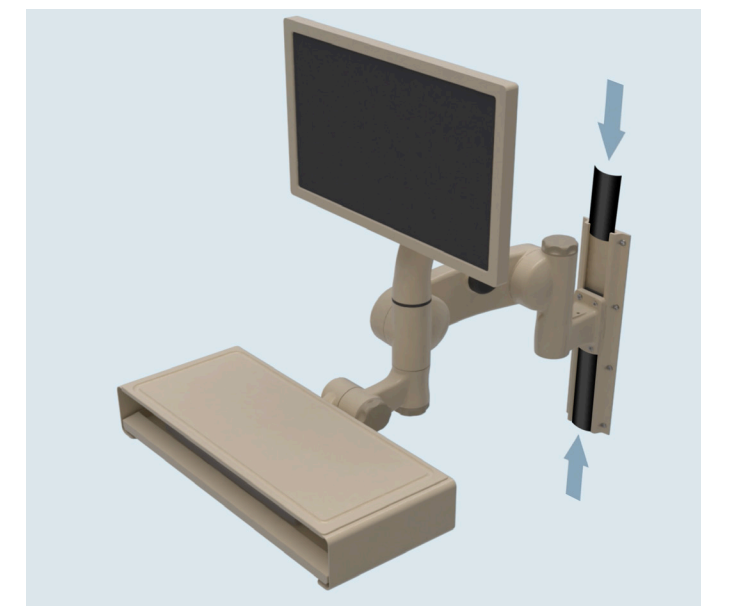

### 7. Install Track Cable Cover


Cut supplied rail cover to desired length(s) and slide into track channels as shown

# CleanMount


## Operation Guide

**STRONGARM**healthcare


Visit [strongarmhealthcare.com](http://strongarmhealthcare.com) to view our complete operations video.

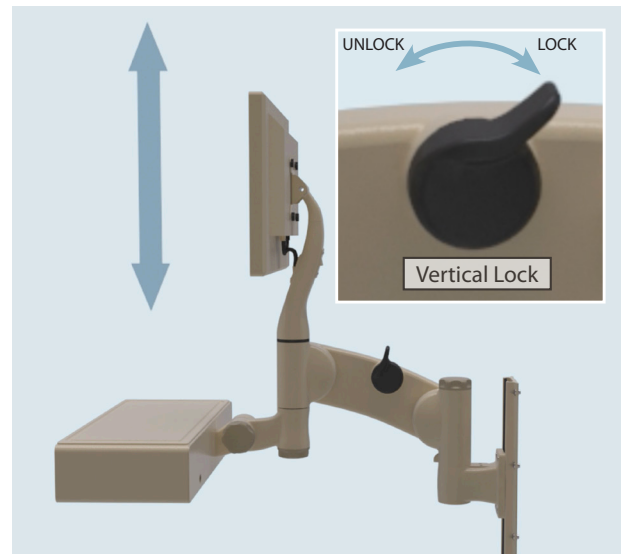


**Vertical Adjustment**

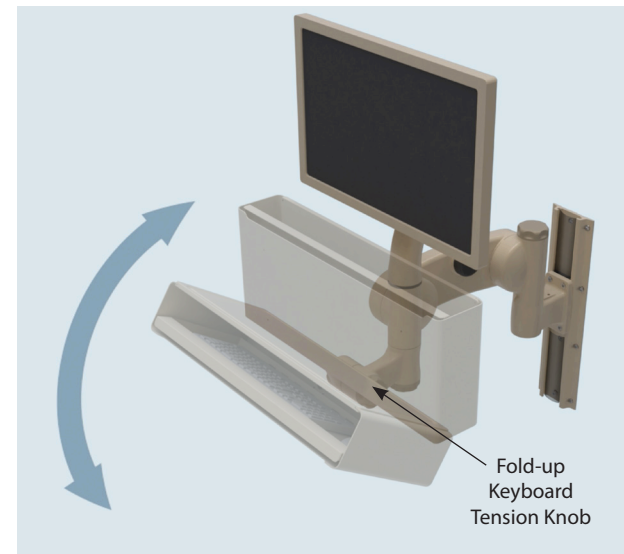

**Fold-up Keyboard**

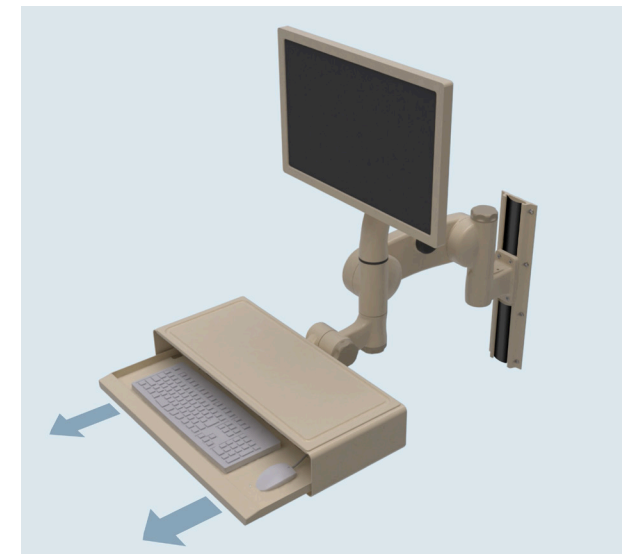


**Pullout Keyboard Tray**

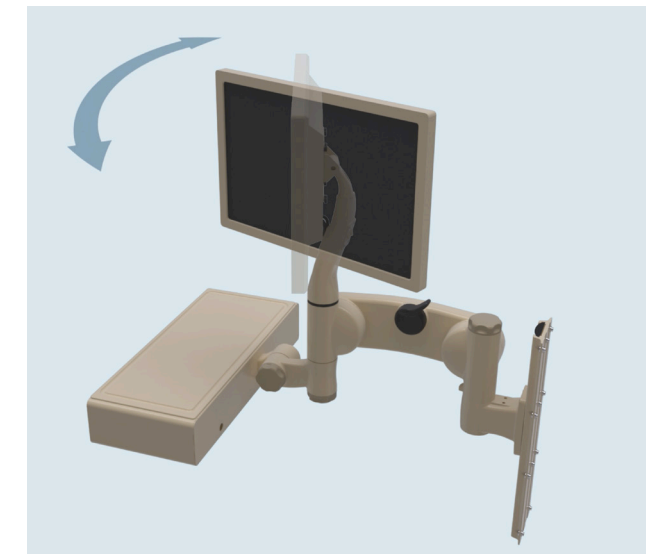


**PatientView\* Display Rotation**


(\*On PatientView equipped models)

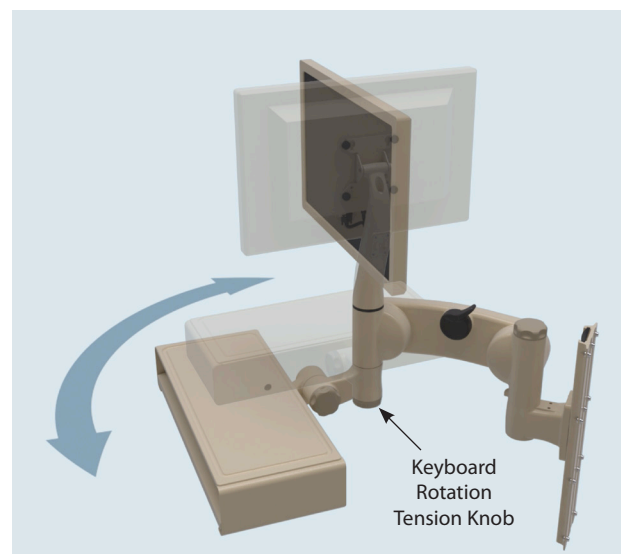


**Keyboard Rotation**

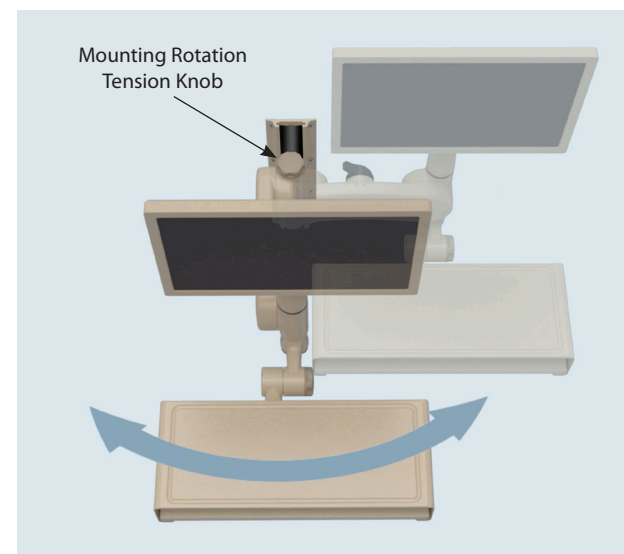

**Mounting Rotation**

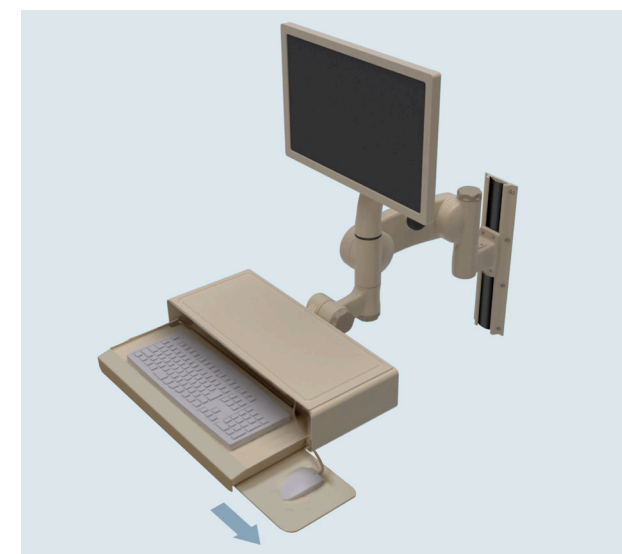

**Slide Out Mouse Tray (optional)**

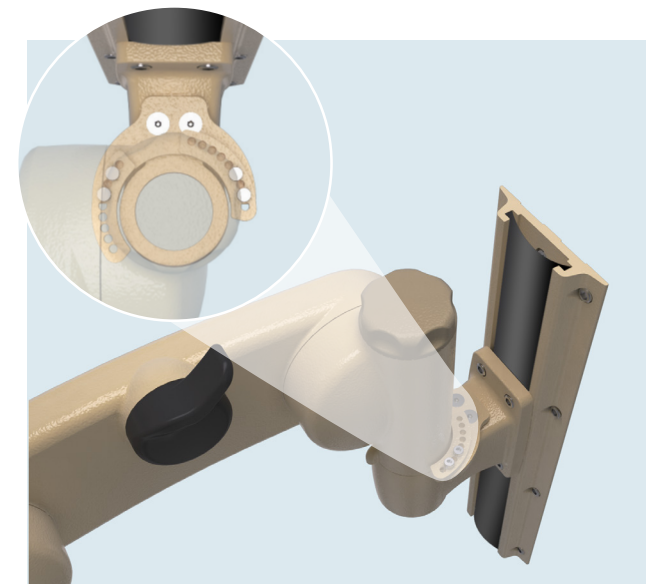

**Rotational Stop (optional\*)**

(\*See instructions included with Rotational Stop for installation and operation guide)