

# MightyMount

## Quick Start Installation Guide

**STRONGARM**healthcare

Visit [strongarmhealthcare.com](http://strongarmhealthcare.com) to view our installation video.

### What's Included

- A) 1/4" Hex Key
- B) (4) Display Thumb Screws (M4 x 8)
- C) (4) Display Thumb Screws (M4 x 12)
- D) Track cover


### Tools You'll Need

- #1 Phillips screwdriver
- Wire Cutter & Tie Wraps (for securing keyboard and mouse cabling: see Step 4 for instructions on accessing prewired USB hub)




### 1. Install Mounting Track\*

\* Refer to track installation instructions

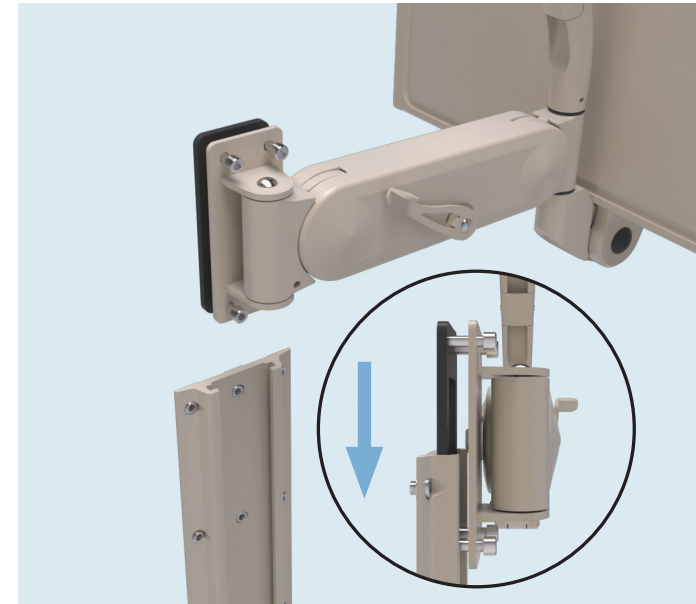

### 2. Install Onto Track

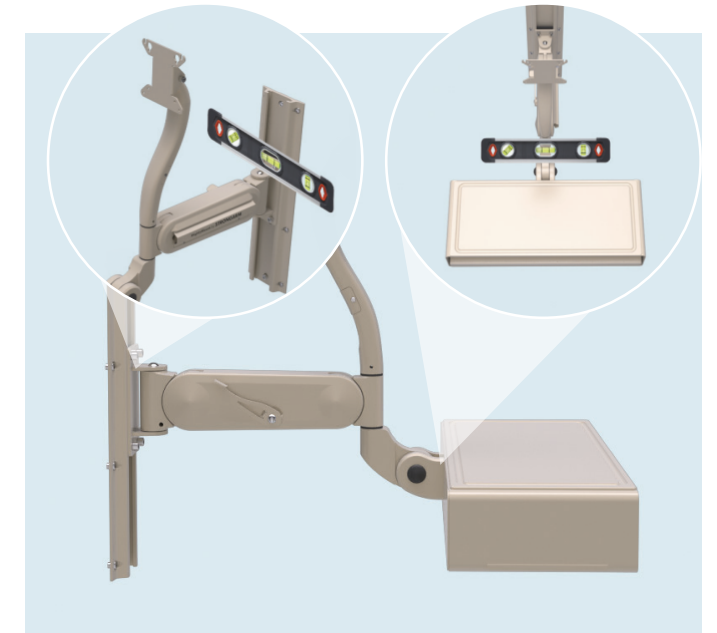


### 3. Level and Tighten

Use 1/4" Hex Key to Tighten Four Slider Key Bolts




### 4. USB Hub Access

- 1) Turn latch on rear of keyboard enclosure to release cover. Pull cover forward to access the USB hub.

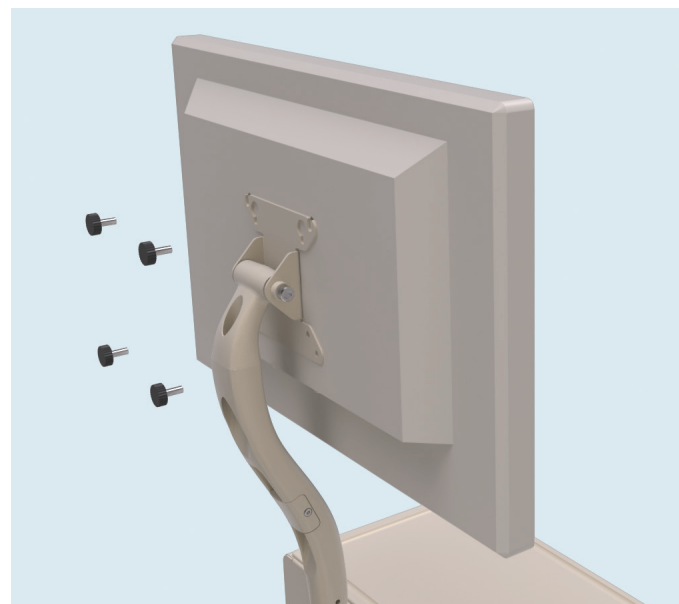


### 5. Install Display

Using (4) M4 x 8 Display Thumb Screws, attach display to MightyMount. (**NOTE:** If the M4 x 8 Display Thumb Screws can't be threaded sufficiently to provide a secure attachment, use the M4 x 12 Display Thumb Screws instead.)

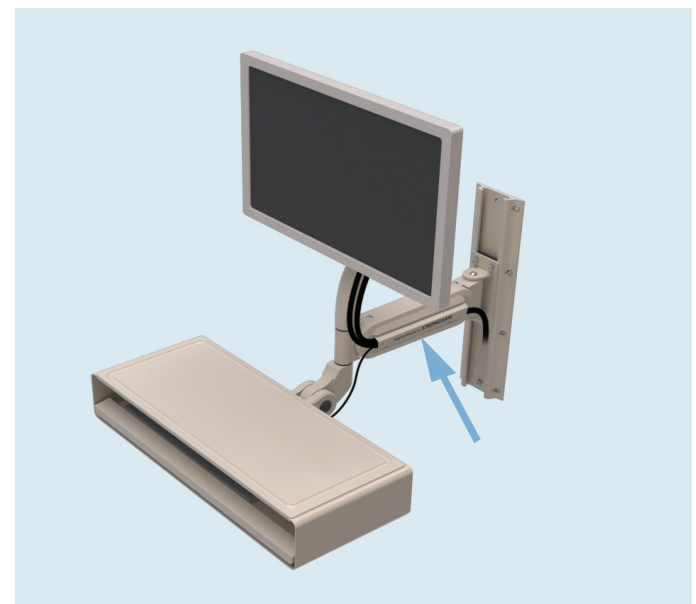

### 6. Run Cables and Install Cable Guard

Display cables can be secured to the neck with a tie wrap fed through the provided holes

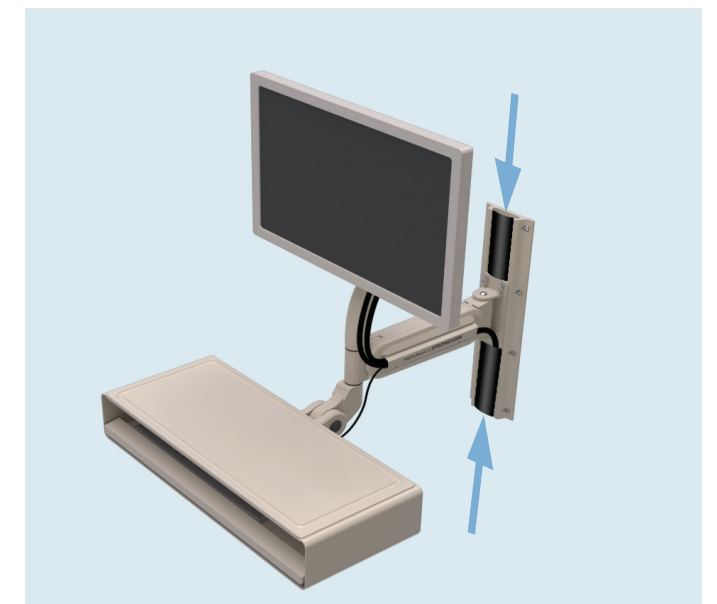

### 7. Install Track Cable Cover


Cut supplied track cover to desired length(s) and slide into track channels as shown

# MightyMount

## Operation Guide

**STRONGARM**healthcare


Visit [strongarmhealthcare.com](http://strongarmhealthcare.com) to view our complete operations video.

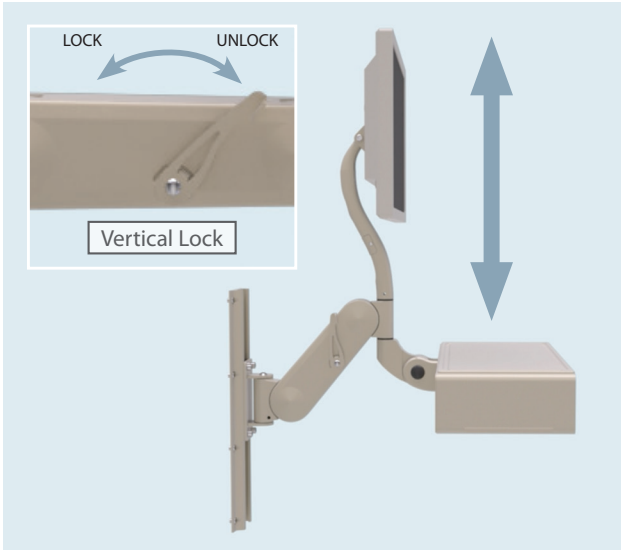


**Vertical Adjustment**




**Mounting Rotation**

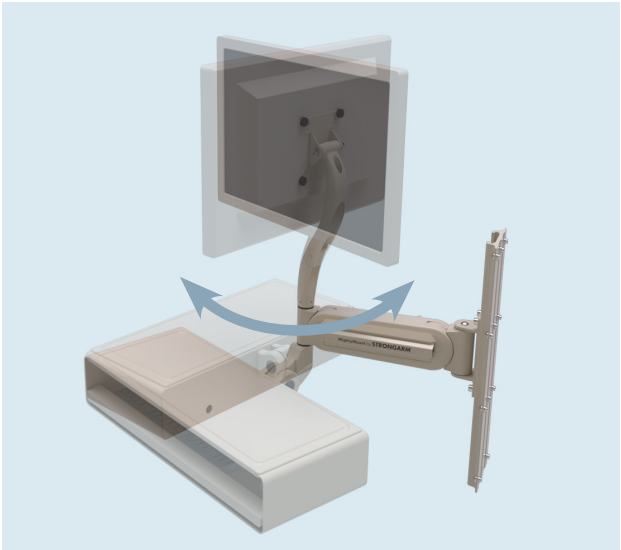

**Display/Keyboard Rotation**

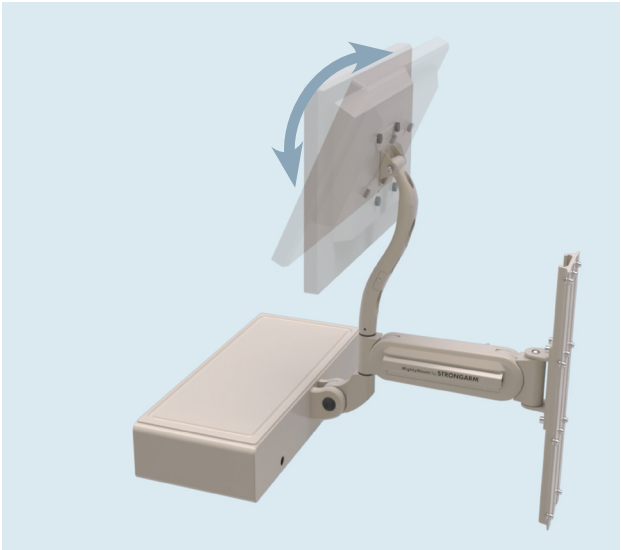


**Display Tilt**

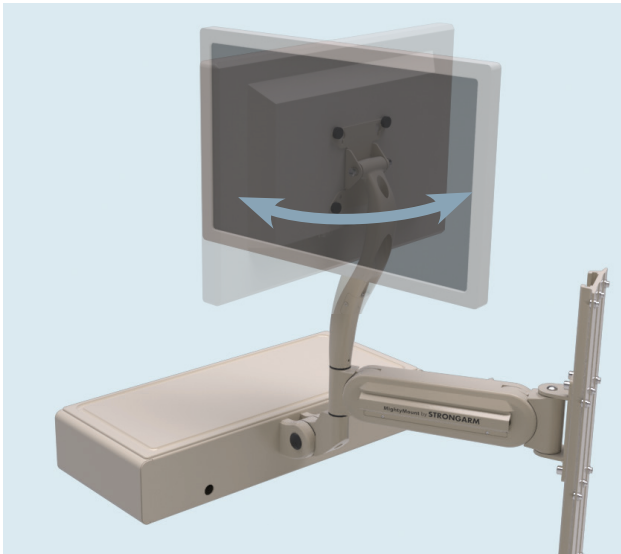


**PatientView\* Display Rotation**


(\*On PatientView equipped models)

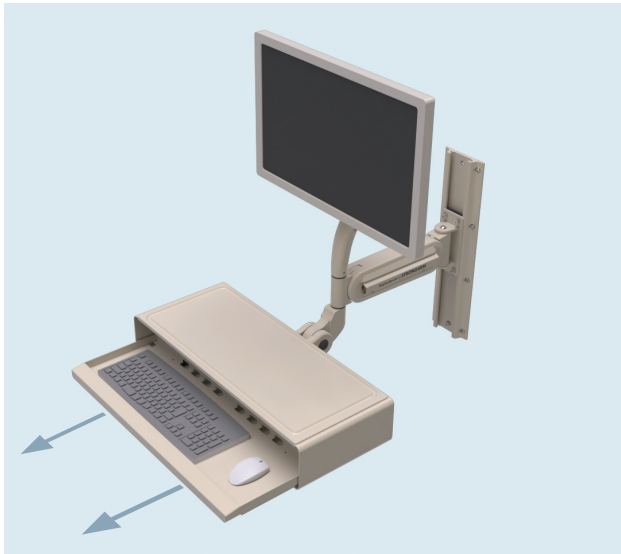

**Pull Out Keyboard Tray**

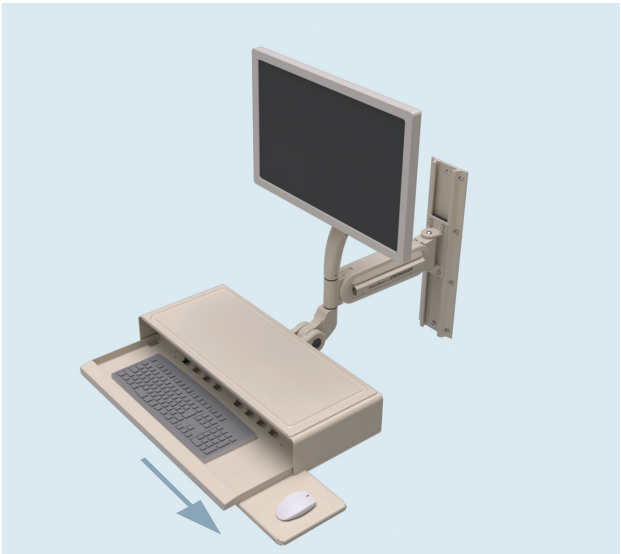

**Slide Out Mouse Tray (optional)**

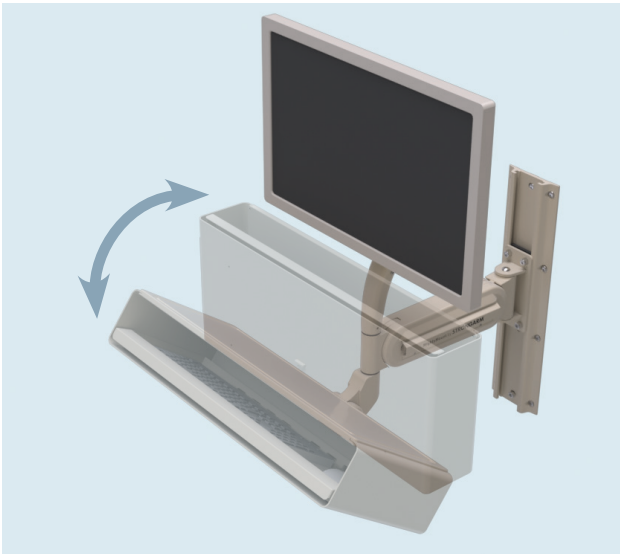

**Fold-up Keyboard**